



Microneedling  
Suggested Working  
Procedure Chart

**CAMPOMATS**  
HUMAN  
TECHNOLOGY



**AREA**  
**FOREHEAD**

**THIN SKIN**  
0.25 - 0.5 mm

**THICK SKIN**  
0.5 - 0.75 mm



**BETWEEN  
EYEBROW**

0.25 - 0.5 mm

0.5 - 1.0 mm



**NOSE**

0.25 mm

0.5 mm



**EYE AREA**

0.25 mm





0.25 - 0.5 mm



**CHEEK BONE**

0.5 mm

0.5 - 1.0 mm

	<b>AREA</b>	<b>THIN SKIN</b>	<b>THICK SKIN</b>
	<b>CHEEK and CHIN</b>	0.5 - 1.0 mm	1.0 - 2.0 mm
	<b>UPPER LIP</b>	0.25 mm	0.25 - 0.75 mm
	<b>FACE SCARS</b>	1.0 - 1.25 mm	1.5 - 2.0 mm
	<b>STRETCH MARKS</b>	1.5 - 2.0 mm	1.5 - 2.0 mm
	<b>SURGICAL &amp; BURN SCARS</b>		2.0 - 2.5 mm

CAMPOMATS Srl

[www.campomats.it](http://www.campomats.it) • [info@campomats.it](mailto:info@campomats.it) • [elisa@campomats.it](mailto:elisa@campomats.it)  
Via Giovanni Falcone 3, 41012 Carpi (MO) ITALY • P.I. 03846110363

